AQUISEL

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AQUISEL VALVE-STOPPER

System available for sample manipulation without the need of uncorking the tube, maintaining it hermetically sealed and avoiding the risk of contamination.

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The AQUISEL system consist of a tube, valve-stopper and an appropriate reagent for specific use.

The AQUISEL tube is made of a high resistance transparent polypropylene. The valve-stopper is an elastic material and **has a special cross-cut design which allows an automatic recover to its initial closed position, after insertion or extraction of the sample.**

The tube's reagent contents is offered through different possibilities identified by the colour of the stopper in accordance with International Standards.

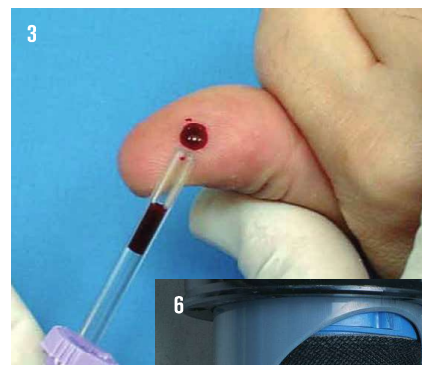
It can be used by most of the current analytical procedures, even if they are manual, semi-automatic or automatic procedures.

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USE AND APPLICATIONS

1. Inserting (blood) samples with syringe (without needle).
2. Extraction and dosage with semi-automatic pipette.
3. Capillary blood extraction.
4. Capillary filling for microhematocrite.
5. Automatic sampling for haematology in primary tubes.
6. Sample road transport.
7. Centrifuge heads adaptable.

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There are different size and volume tubes with additives: anticoagulant, antiglycolitics, clotting accelerators or empty tubes.

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Suitable for paediatrics or small sample volumes.

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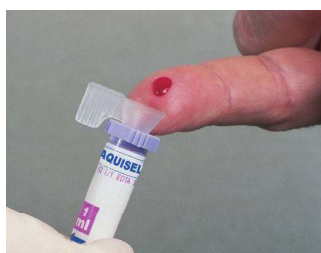
The tubes are presented in polystyrene racks of 100 units. Every 50 units are covered with a transparent lid for protection.

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Funnel shaped adapter designed to allow blood collection inside the tube, extracted by capillary puncture.

It also allows the AQUISEL valve cap system to be kept open when an inner manipulation is required without having to be unblocked.

Once the CAP GALET has been extracted, the tube remains closed again for its conservation or disposal.

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Tubes for pediatrics and for low blood volumes (up to 1 ml)

Tube dimensions: 11x40 (ØxL)

Upper ring that allows to enter a 13x75 tube to be centrifuged.

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Accessory that together with the AQUISEL tubes for low blood volumes, allows the collection of blood extracted by capillary extraction directly into the tube. The capillary effect facilitates collection of blood drop by drop. Once the capillary is removed, the tube is closed again thanks to its valve-stopper.

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It is a funnel-shaped accessory that, together with AQUISEL® tubes for low blood volumes, allows the collection of blood extracted by capillary extraction directly into the tube. Once the Mini Galet is removed, the new tube remains closed thanks to its valve-stopper.

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Blood extraction Blood-Gas/Capillary.

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AQUISEL PIPETTE

PIPETTES FOR ERYTHROCYTE SEDIMENTATION READ E.S.R. WESTERGREN METHOD.

Pipettes made in polystyrene with similar properties as glass ones.

The ZERO-CLOSURE system allows the pipette filling, preventing that the blood exceeds the zero point and also avoiding total or partial loss. Automatic sealing makes it a disposable product.

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AQUISEL Pipette A-2 without rubber.

For any kind of tube use, even with AQUISEL or with a conventional stopper.

Suitable for mechanical or manual aspiration.

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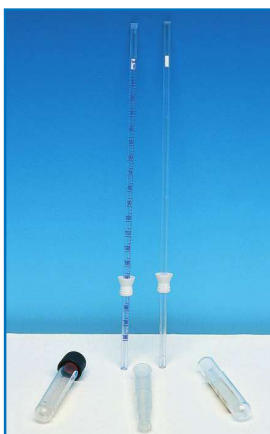
AQUISEL Pipette P-2 with rubber.

For 75x13 tubes with AQUISEL or with a conventional stopper.

Pipette filling by pressure till the bottom of the tube.

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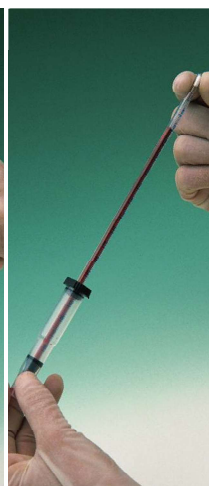
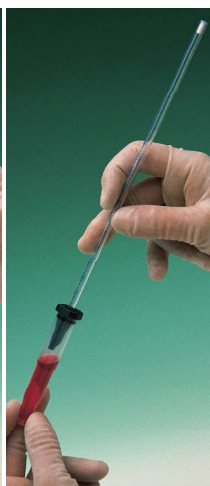
AQUISEL Pipette P-3 and P-4 MICRO systems

Set of AQUISEL pipettes and tubes. Tubes come with the valve-stopper which allows the pipette insertion without the need of uncorking.

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AQUISEL Pipettes A-2

Expanded polystyrene packaging of
500 units.

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AQUISEL Pipettes P-2

200 units cardboard package.
Supplied in 800 units boxes.

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Set P-3. AQUISEL Pipette and tube

200 units cardboard package and 100
tubes rack made of expanded polysty-
rene, supplied in 600 sets boxes.

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
Set P-4 Micro. AQUISEL Pipette and tube

100 pipettes cardboard package and 100
tubes rack made of expanded polysty-
rene, supplied in 400 sets box.

MODELO MODEL	CÓDIGO CODE	Nº UNIDADES CAJA UNITS / BOX	PESO CAJA Kg BOX WEIGHT Kg	MEDIDAS CAJA cm - BOX DIMENSIONS cm		
				Ancho Width	Fondo Depth	Alto Height
A-2	1750224	500	1,5	27	14	11
P-2	1780225	800	4	55	27	26
Set P-3	1760200	600 pipetas + 600 tubos	6	47	43	26
Set P-4 Micro	1740100	400 pipetas + 400 tubos	3,6	20	40	37

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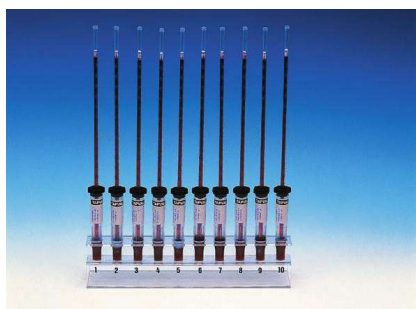Metallic Westergren stand

10 place autoclavable stand.

For any kind of AQUISEL pipettes with Ø 13 mm
tubes and A-2 pipettes with Ø 16 mm tubes.

Easy and functional system that allows the total
set disposal.

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BOROSILICATE PIPETTE

FOR THE DETERMINATION OF THE GLOBULAR SEDIMENTATION SPEED ESR

Westergren model. Manufactured in sterilizable borosilicate graduated from 0 to 20 cm.

CÓDIGO CODE	Ø interior mm Interior Ø mm	Ø Exterior mm Exterior Ø mm	Largo mm Large mm
1001943	3	7	300

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AQUISEL URINE CONTAINER

CONTAINER FOR URINE SAMPLES COLLECTION WITH AQUISEL VALVE-STOPPER.

The AQUISEL VALVE-STOPPER allows the urine manipulation without uncorking, eliminating the contamination risks, samples loss, spillage, odours, etc.

All the AQUISEL urine container presentations will be supplied with the GALLET accessory, which makes easy the sample dosage to a tube for sedimentation analysis, or the urine test strip insertion.

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Insertion of the GALET in the AQUISEL VALVE-STOPPER.

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Urine sample extraction through the GALET.

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Dosage through automatic or conventional pipette.

CÓDIGO CODE	BOLSA BAG	MONTADO ASSAMBLED	UDS. CAJA UNITS BOX	PESO Kg WEIGHT Kg	MEDIDAS cm - DIMENSIONS cm					
					Ancho Width	Fondo Depth	Alto Height	Alto Height	Alto Height	Alto Height
1660223	NO NO	NO NO	600	12	60	40	43			
1640234	NO NO	SI YES	400	8	60	45	43			
1630233	SI YES	SI YES	300	7	60	45	43			

Código **1600230** GALET CONTENEDOR (paquetes de 500 unidades).
Code **1600230** GALET CONTAINER (500 units package).

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AQUISEL URINE CONTAINER

CONTAINER FOR URINE SAMPLES COLLECTION WITHOUT AQUISEL VALVE-STOPPER.

Watertight, made of polypropylene. Graduated and transparent.
Individual sterilized bagging

CÓDIGO CODE	BOLSA BAG	UDS. CAJA UNITS BOX	CAPACIDAD ML CAPACITY ML	MEDIDAS cm - DIMENSIONS cm		
				Ø	Ø	Alto Height
1951300	SI YES	250	100	60	73	
1951301	SI YES	300	60	53	65	

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FIXATION AND CONCENTRATION OF FAECAL PARASITES “Aquisfek-2”

New compact design of Aquisfek-2 device, which allows an easy and convenient sample collection, a safe transportation with an easier handling system and a more effective filtration.

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Note: For centrifugation, Cencom 2 is recommended. Part number 7002240. See page 146

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SAMPLE COLLECTION.

EASY TO HANDLE BY THE PATIENT.

Separate the two parts (A and B) of the device and with the aid of the spoon built-in at the end of the filter, collect the sample.

Introduce the spoon with the sample back into the liquid and screw to close.

PROCESSING THE SAMPLES IN THE LABORATORY.

FAST AND EASY.

Once the sample has been homogenized, the whole device is centrifuged.

After centrifugation, discard the part A (spoon) and with the aid of a pipette, remove the supernatant in the tube (Part B).

The concentrated sample residue is transferred to a slide for a further microscopic observation.

TWO DIFERENT FORMATS

KIT 1. With formalin / acetate. Parasite common fixer.

KIT 2. Without formalin to minimize the risk of environmental contamination.

KIT 1. With formalin.

Part number

1800508 99 units supplied (33 bags of 3 units).

1800505 100 individual units supplied.

KIT 2. Without formalin

Part number

1800503 99 units supplied (33 bags of 3 units).

1800500 100 individual units supplied.

CONTAINER FOR FECAL AND MISCELLANEOUS SAMPLES

CONTAINER FOR THE COLLECTION OF FECAL SAMPLES

Made of polypropylene. Opaque, with spoon for sampling.

Individual sterilized bagging.

CÓDIGO CODE	BOLSA BAG	UDS. CAJA UNITS BOX	CAPACIDAD ML CAPACITY ML	MEDIDAS cm - DIMENSIONS cm Ø Ø Alto Height
1951302	SI YES	300	60	48 47