

1. Instruction for use

(1) Choosing the right sterilization roll according to the size of sterilizing items. Then put clean and dry items into the sterilization roll. Sterilized items should not exceed 3/4 of the length of sterilization pouch, so as to ensure adequate sealing, otherwise the pouch is easy to burst.

(2) The instrument should be placed in the opposite direction with peeling side to ensure safety.

(3) Sealing sterilization pouch by using sealing machine, marking sterilization time, content and usage information.

(4) Sealing the paper side with film side. The sealing temperature is 200-230°C, sealing pressure is 0.15kg/cm² and lasting time is 4-6S.

(5) Put the sterilization pouch together after sealing as paper to paper side. Then put the items into Steam or ETO sterilizer equipment to sterilize.

(6) Check indicator mark color change process: Before steam sterilization, color is blue, it will change into brown after sterilized. Before ETO sterilization, the color is red, it will change into yellow after sterilized.

(7) Place the sterilized sterilization pouch in dry, dark and clean area for storage and stand-by use.

(8) Peel the pouch as the pointing direction when use and then take out the items.

2. Suggestion

(1) After sterilization, the pouch should be stored in qualified place. It will be better to be used in 6 months after steam sterilization and in 24 months after ETO sterilization.

(2) Please check the sterilization pouch before use. Do not use if package is damaged.